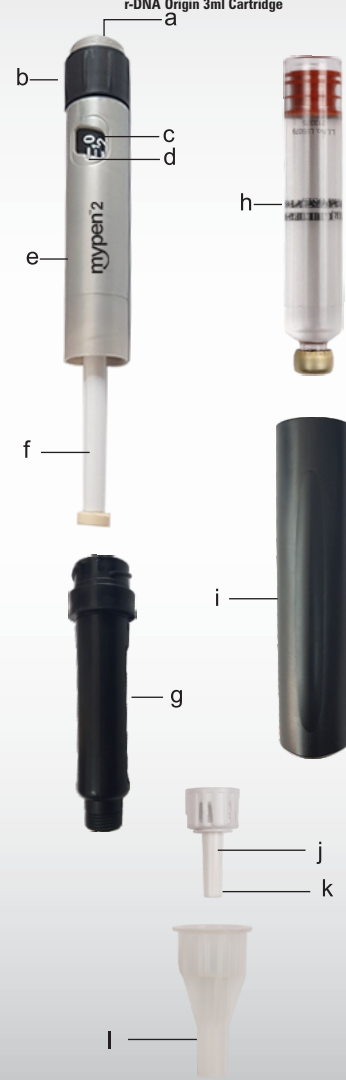
mypen[®]2

Insulin Delivery Device

Companion for Diabetes Control

To use with Insulin Injection I.P. 100 IU/mL (Wosulin[®] R)
Isophane Insulin Injection I.P. 100 IU/mL (Wosulin[®] N)
Biphasic Isophane Insulin Injection I.P. 100 IU/mL (Wosulin[®] 30/70)
Biphasic Isophane Insulin Injection I.P. 100 IU/mL (Wosulin[®] 50/50)
Insulin Glargine Injection I.P. 100 IU/ml (Glaritus[™])
Insulin Glargine Injection I.P. 100 IU/ml (Glarzen[®])
r-DNA Origin 3ml Cartridge

COMPONENTS (घटक)

- | | |
|------------------------|--------------------------|
| a. Plunger | a. प्लंजर |
| b. Dose Dialer | b. खुराक डायलर |
| c. Dose display window | c. खुराक प्रदर्शन खिड़की |
| d. Dose Indicator | d. खुराक संकेतक |
| e. Pen Body | e. पेन बॉडी |
| f. Internal Plunger | f. आंतरिक प्लंजर |
| g. Cartridge Holder | g. कार्टिज धारक |
| h. Cartridge | h. कार्टिज |
| i. Pen Cap | i. पेन का ढक्कन |
| j. Needle* | j. सुई * |
| k. Needle Cap* | k. सुई का ढक्कन * |
| l. Protective Cap | l. सुरक्षात्मक ढक्कन |

*Needle, Cartridge and protective caps (not included with this pack; the needle details are provided for instructions of use purpose only)

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mypen[®]2 is an easy-to-use insulin delivery pen designed to administer insulin from a 3mL Wosulin[®] / Glaritus[™] Cartridge / Glarzen[™] Cartridge.

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mypen[®]2 can deliver doses from 1-60 units in increments of 1 unit.

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Please read these step-by-step instructions carefully before using mypen[®]2.

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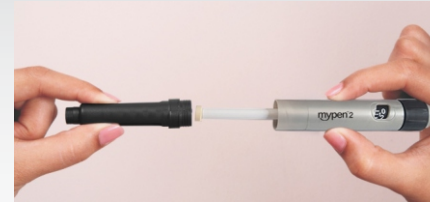
Before using needle kindly check the expiry date printed on the needle pack.

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
If using (Wosulin[®]30/70 / Wosulin[®]50/50 / Wosulin[®]N) you will need to resuspend the insulin by turning the cartridge up and down or rolling between the palms gently.

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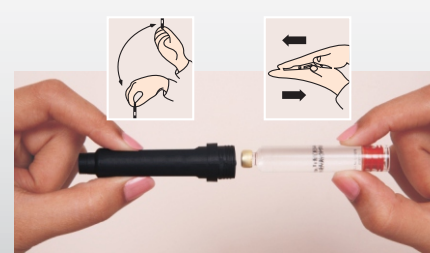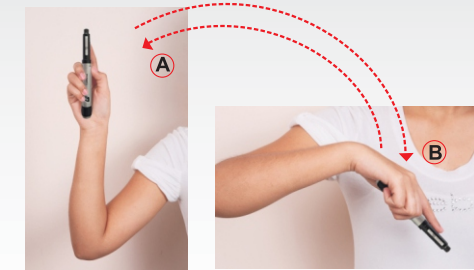
If using (Wosulin[®]30/70 / Wosulin[®]50/50 / Wosulin[®]N) turn the pen up and down at least 10 times or rolling between the palms gently. Then attach the needle to the pen cartridge holder.

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It is important to Prime mypen[®]2 before use to remove any air that may be inside the needle.

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3

4

9. Push the plunger completely and the dose indicator will return to 0 (Zero). A drop of insulin appears at the tip of the needle.

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Repeat steps from 7 to 9 till a drop of insulin appear at the top of the needle; ensuring explosion of air bubbles.

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10. Make sure the dose indicator points to 0 (Zero) units. Dial the required amount of insulin by rotating the dose dialer. The figure that appears in the front of the dose indicator indicates the amount of insulin dialled by you.

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In case you dial more than the recommended units; simply dial the dose dialer in the opposite direction to the desired dose unit.

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"mypen[®]2 meets the dose accuracy and functional requirements of ISO 11608-1 with Wockhardt's 3 mL Cartridge".

5

11. Insert the needle at the site of injection using the insulin technique recommended by the Doctor. Push the plunger completely.

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12. Count till 10 and remove the needle from the skin. Reattach the white outer protective needle cap and remove the needle. Replace the pen cap onto mypen[®]2.

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Before making subsequent injection always make sure that a new pen needle is used, the pen is primed and a drop of insulin is present at the tip of the needle as mentioned in steps 5 to 9.

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13. If there is not enough insulin in the cartridge for the whole dose, you will be able to see the number of units you still need to inject in the dose indicator. You must always check this after you have made the injection.

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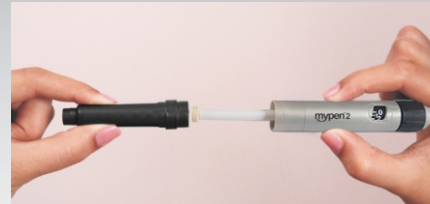
CHANGING CARTRIDGE (कार्ट्रिज बदलना)

14. To change the cartridge unscrew and remove the cartridge holder from the pen body. Remove the cartridge carefully.

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6

15. To retract internal plunger hold the internal plunger firmly and rotate the pen body clockwise till the internal plunger is fully retracted. Internal plunger can also be retracted by pushing the round plunger cap with thumb pressure.

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Follow the step 3 for loading the cartridge.

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Storage: The mypen[®]2 should always be stored or carried without needle and the pen cap on. To clean the mypen[®]2 wipe the body with a clean damp cloth. Do not keep mypen[®]2 in refrigerator in between the injections.

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7

Important safety precautions

Failure to follow these instructions may result in an inaccurate dose.

- Always check before each injection that the insulin cartridge in mypen[®]2 contains the correct amount of the insulin to be used.
- Always expel any air with the needle pointing upwards before each injection.
- The needle should be removed and safely disposed immediately after each injection dose and store the pen without a needle. This prevents contamination, infection, leakage of insulin, and will ensure accurate dosing.
- Your Doctor has prescribed the type of insulin that he/she believes is best for you. Do not use another insulin except on the advice and direction of your Doctor.
- Please check the contents of the package as listed on the outside of the box.

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© Regd. Trademark of Wockhardt Patented Technology by Wockhardt Limited in India

WOCKHARDT

Manufactured by
WOCKHARDT LIMITED
Biotech Park, H-14/2, MIDC, Waluj,
Aurangabad 431136,
Maharashtra state, India

For more details on Diet & Diabetes visit
www.wockhardt diabetic.com

mypen[®]2
Insulin Delivery Device
Companion for Diabetes Control

WOCKHARDT

L. I. No. ms-5366/1
777777

CMYK

Leaflet:

Open Size: 350 x 180mm (L x H)

Folded Size: 70 x 180mm (L x H)